

FireCR

Maintenance Guide

Veterinary Dental Cassette Kit

Doc No. : TM-723-EN

Rev 1.2.1 Nov 2014

Part No. : CR-FPM-03-002-EN

3DISC, FireCR, Quantor and the **3D Cube** are trademarks of **3D Imaging & Simulations Corp.**, South Korea, and its affiliates. All other trademarks are held by their respective owners and are used in an editorial fashion with no intention of infringement. The data in this publication are for illustration purposes only and do not necessarily represent standards or specifications, which must be met by **3D Imaging & Simulations Corp.** All information contained herein is intended for guidance purposes only, and characteristics of the products and services described in this publication can be changed at any time without notice. Products and services may not be available in your local area. Please contact your local sales representative for availability information. **3D Imaging & Simulations Corp.** strives to provide as accurate information as possible, but shall not be responsible for any typographical error.

© Copyright 2014 **3D Imaging & Simulations Corp.** All rights reserved.

3DISC
I M A G I N G

Contact



3D Imaging & Simulations Corp.
Bldg.1, 48, Yuseong-daero 1184 beon-gil,
Yuseong-gu, Daejeon, 305-345 Korea

Tel : 82-42-931-2100
Fax : 82-42-931-2299

Website : www.3DISCimaging.com
E-mail : info@3DISCimaging.com

3DISC Americas
22560 Glenn Dr, Suite 116
Sterling, VA 20164 USA
Tel : 1-703-430-6080
E-mail : info@3DISCimaging.com



3DISC Europe
Gydevang, 39-41, 3450 Alleroed, Denmark
Tel : 45-88-276-650
E-mail : info@3DISCimaging.com

Warnings and Used Symbols

To ensure the safety of patients, staff and other persons, any changes to software and hardware delivered by **3D Imaging & Simulations Corp.** may only be made with prior written permission from **3D Imaging & Simulations Corp.**

Please read the respective manuals of the connected software, such as acquisition and diagnostic software, before starting to use the system.

The following symbols will be used throughout this manual:



DANGER

General prohibition indication.

The functionality of the system can be destroyed in the case of incorrect use.

If unauthorized changes have been made to delivered system and accessories, the warranty by **3D Imaging & Simulations Corp.** becomes void. **3D Imaging & Simulations Corp.** will not accept any responsibility or liability for the proper functioning of the product in such a case.



DANGER

General mandatory action manual.

The functionality of the system can be destroyed in the case of incorrect use.

If unauthorized changes have been made to delivered system and accessories, the warranty by **3D Imaging & Simulations Corp.** becomes void. **3D Imaging & Simulations Corp.** will not accept any responsibility or liability for the improper functioning of the product in such a case.



WARNING

The functionality of the system can be limited in the case of incorrect use. Hints that require special attention.



NOTE

Notes represent information that is important to know but which do not affect the functionality of the system.

General Safety Guidelines

All the safety and operating instructions should be read carefully before this device is operated.

This device has been designed and tested to meet strict safety requirements applicable to medical equipment, and has been supplied in a safe condition. To ensure personnel and patient safety, the device shall be operated and serviced in compliance with all procedures, warnings and precautions during all phases of operation and service of this device. Failure to comply to with safety guidelines may result in injury to service personnel, operator, or patient. **3D Imaging & Simulations Corp.** assumes no liability for failure to comply.

If this device is not used as specified, the protection provided by the device could be impaired. This device must be used in a normal condition only.

Installation, service and operation of this device should only be undertaken by qualified trained personnel. The operator should study instructions and precautions carefully before starting to use the device listed here and throughout the manual.

There are no user serviceable parts inside this device. The device should only be opened and serviced by qualified service personnel. Failure to heed this warning may result in injury to service personnel or damage to equipment, and void any and all warranties. If there is a service problem, please contact **3D Imaging & Simulations Corp.** or authorized dealer.

Do not spill liquids on the device, and never operate the device in a wet environment.

Keep the device from radiators and heat sources.

Use the device only with accessories supplied with this device.

This device is intended to be grounded. Plug power cord into properly grounded electrical outlets. This cord is equipped with three-prong plugs to help ensure proper grounding.

This device contains static sensitive components. Proper static handling procedures and equipment must be used when servicing this device.

Do not look inside of the device.


If any of the following conditions occur, unplug the device from the electrical outlet and contact authorized service personnel.

- ✓ The power cord or power adapter is damaged.
- ✓ An object has fallen into the device.
- ✓ The device has been exposed to water.
- ✓ The device has been dropped or damaged.
- ✓ The device does not operate correctly when the operating instructions are followed.


1. Veterinary Dental Cassette

Supporting multi imaging plates on a moving plate.

- ✓ Veterinary Dental Cassette
 - Imaging plate size : Each of imaging plate size is marked on the moving plate
 - (1) Size 0 (22x31 mm) - 2ea,
 - (2) Size 1 (24x40 mm) – 2ea,
 - (3) Size 2 (31x41 mm) – 4ea,
 - (4) Size 3 (27x54 mm) – 2ea
 - (5) Size 4c (48x54 mm) - 2ea,
 - (6) Size 4 (76x57 mm) – 1ea

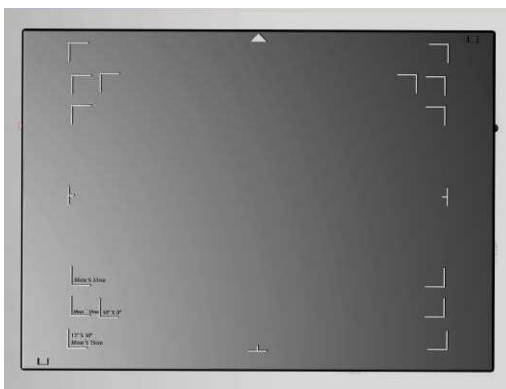

Veterinary Dental Cassette

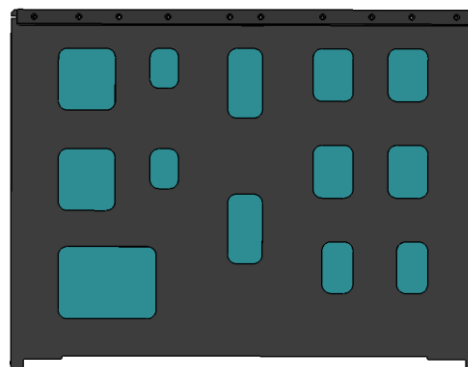

Moving Plate

2. Imaging Plate Cleaning

- ✓ Protect your investment

Dust and dirt are piled up on CR imaging plate, leading to image artifacts. To avoid contamination, plates need cleaning at regular intervals.



WARNING

As CR imaging plates are highly sensitive to water based cleaning fluids, we strongly recommend an ethanol-based plate cleaner.

If possible, use a cleaner that is specially designed for CR imaging plates. This will help to ensure the longest possible service life.


2.1. Service Life

Imaging plates in the following condition cannot be used.

- ✓ Scratches on the surface of the protective layer and they appear on the image.
- ✓ The phosphor layer has peeled away from the support layer.

2.2. Imaging Plate Cleaning Tools


- ✓ Moving plate extracting tool

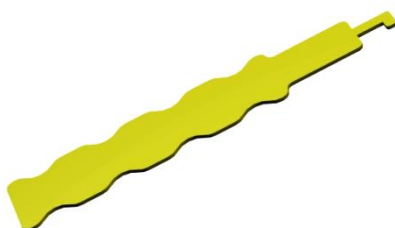


- ✓ Imaging plate cleaner
Use ethanol based cleaner : ethanol > 99.5%
- ✓ Soft lint free dry cotton (nonabrasive cloth)

2.3. Imaging Plate Cleaning Process


2.3.1. Moving Plate Pull Out

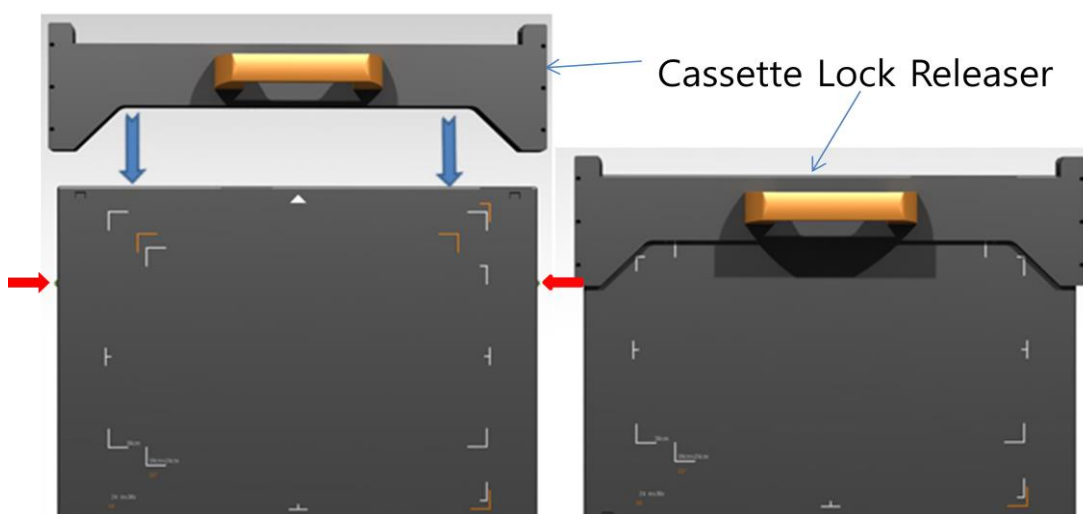

There are 2 buttons on each side of cassette for unlocking the moving plate. See the red arrows. To release the Moving Plate from the Cassette, the Cassette Lock Releaser should be linked up with the Cassette. The the Cassette Lock Releaser are pressing those buttons during it is linked up with the Cassette correctly

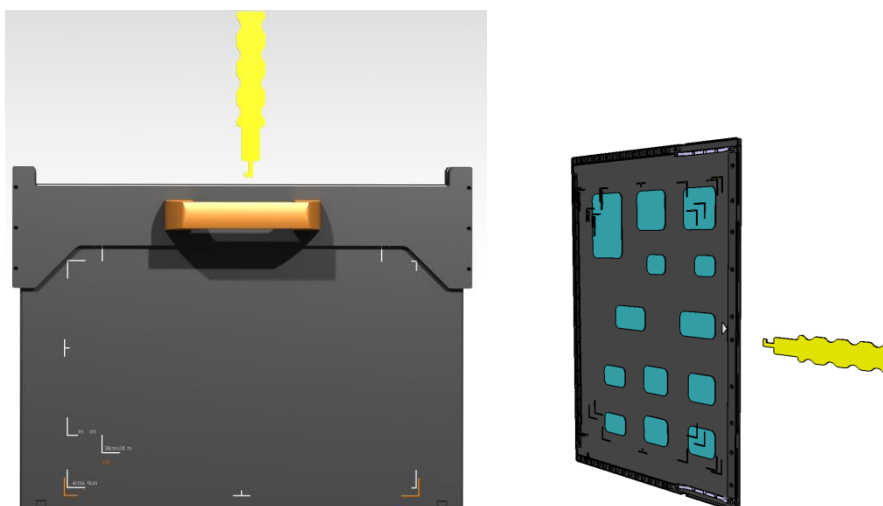



Stand the Cassette on the side and press the buttons using the Cassette Lock Releaser. Make sure both of buttons are pressed when you want to release the Imaging Plate from the Cassette.

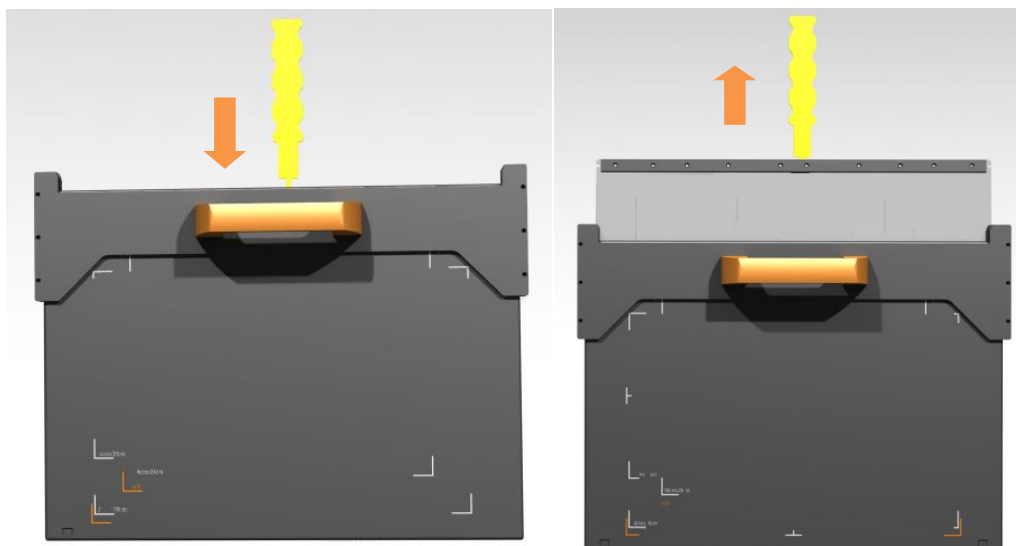

Hook and pull the moving plate out.

2.3.2. Cleaning Process

- ✓ Using the dry cleaning cloths listed previously, clean the imaging plate surface as illustrated below.

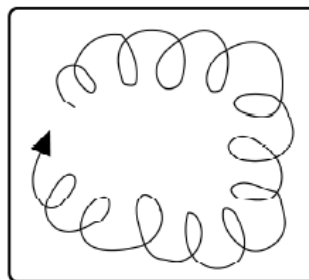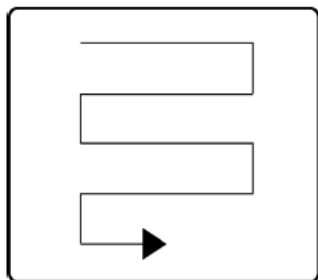

- ✓ Use ethanol dampened cleaning cloths to clean stains that do not come off with the above method.
- ✓ When the imaging plate surface has been cleaned using ethanol dampened cloth, slightly wipe it off with the listed dry cloths.

**WARNING**

Do not pour ethanol directly on to the imaging plate.
Make sure the imaging plate surface is completely dry before using the imaging plate.


**WARNING**

Be sure not to damage the imaging plate.
Do not scratch the imaging plate with your fingernails.
Never use erasures or any other solvents.
Never use hand lotions or sunblock cosmetics prior to handling the imaging plate.
Do not bend or crack the imaging plate during use or cleaning.
Do not stack other objects on the image plate.
Do not let the imaging plate stand under direct rays (ultraviolet rays).

**WARNING**

Imaging plates must be discarded according to laws and regulations applicable in each country.

2.3.3. Insert Moving Plate

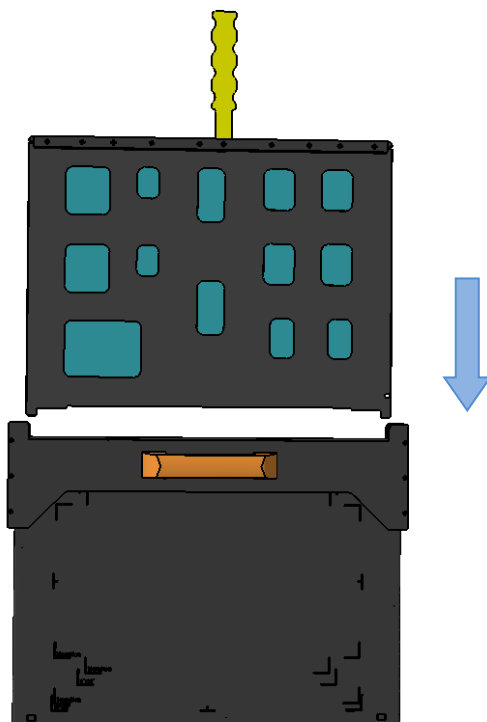

Stand the cassette on the side and press the buttons with the Cassette Lock Releaser. Make sure both of buttons are pressed. Insert moving plate into the cassette



WARNING

Make sure both of buttons keep pressed firmly.




WARNING

Do not put the imaging plate upside down when it is inserted.
Do not insert the moving plate in wrong direction.
Do not lean the moving plate when it is inserted, the Imaging Plate can be scratched by the Cassette.



DANGER

Never press push buttons of the cassette with upside down position. A crash will cause the damage of the imaging plate and moving plate which can affect X-ray image quality.

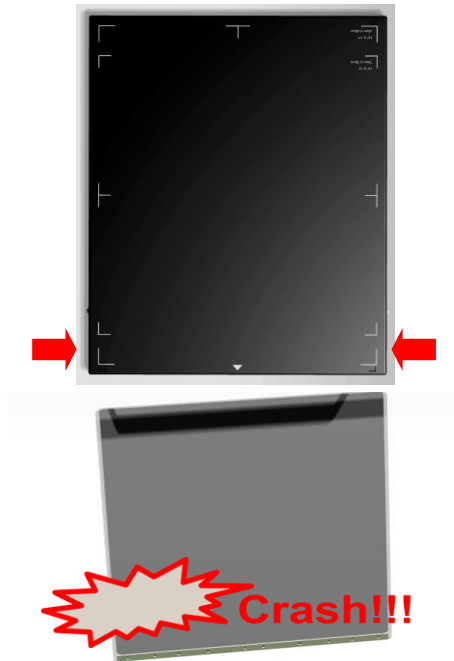

Always carry the cassette with the slot side up.

